

**Warning: For your safety, please observe the following Safety Precautions when using the DPL™ System:**

- ◆ Read all instructions and safety precautions before using the DPL™ system.
- ◆ The DPL™ System is an electrical device and caution must be used when using this unit.
- ◆ This device operates at 110V AC and 60 Hz only. Connect to a properly grounded outlet only.
- ◆ Use indoors only. Use only as described in the user manual.
- ◆ Unplug from outlet when not in use.
- ◆ Do not allow the device to be used as a toy. Close attention is necessary when device is operated near children.
- ◆ Do not use if cord or plug appears to be damaged. If the device is not working properly, or has been dropped, damaged, left outdoors, or dropped into water do not use.
- ◆ Do not pull, lift, or carry by the power cord. Do not use the cord as a handle, close a door on the cord, or pull the cord around or against sharp edges or corners.
- ◆ Keep the cord away from heated surfaces.
- ◆ There are no user-serviceable parts inside the unit. Any attempt to open the unit will void the warranty and could lead to serious damage to the unit and/or personal injury to the operator.
- ◆ Turn off all controls before unplugging unit from outlet.
- ◆ Do not unplug by pulling the cord. Grasp the plug itself, not the cord, when unplugging from an outlet.
- ◆ Do not handle the plug or device with wet hands or apply next to wet skin.
- ◆ Do not operate the unit in areas where flammable or combustible products or vapors may be present.
- ◆ Do not sleep in front of device with the device on.
- ◆ Clean only with soft brush or cloth. NEVER immerse or wash unit with water.

## MEDICAL PRECAUTIONS

- ◆ Consult your doctor prior to using.
- ◆ If you experience any adverse effect or have a concern regarding use, stop use immediately and contact your doctor.
- ◆ If pregnant, please consult your physician, prior to use.
- ◆ If user has been diagnosed or suspects thyroid problems do not use over thyroid gland (the neck area).
- ◆ Do not use if you are taking drugs known to cause photosensitivity reactions.
- ◆ Consult your doctor or dermatologist before using device over any suspicious or cancerous lesion.
- ◆ Do not use over or near a recent steroid injection.
- ◆ Individuals using topical or systemic steroids or N.S.A.I.D.'s for pain or skin conditions may experience mild "flare up" of their symptoms and should discontinue DPL™ therapy.
- ◆ Do not apply light to an open wound, as heat may stimulate circulation and cause bleeding.
- ◆ DPL™ Therapy is not to be used by incapacitated, sleeping or unconscious persons.
- ◆ Do not leave small children unattended close to unit.
- ◆ Do not allow children to use - this is not a toy.
- ◆ Do not use on infants.
- ◆ Do not use with liniments, salves or ointments that contain heat-producing ingredients, as a skin burn could result.
- ◆ If the DPL™ System begins to overheat turn unit off immediately.
- ◆ If using the unit causes any discomfort, stop use immediately.
- ◆ Individual areas of the skin should not be exposed to more than two 9-minute treatments within any 24 hour period.